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Creating an Instant, Perfect and Clean Softening in Uniformity and Adequate Temperature !!

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Computerized Occlusion Rim Softening Equipment

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This equipment, DT-200D, is to soften the wax occlusion rim applied in the bite control in the process of fabricating the dentures for the patients with edentulous jaws. This equipment can easily soften the wax instantly (approx.in 10 sec.) biting to an optional depth from the top surface of the occlusion rim to solve the problems of smoke and smell in the clinic office and to serve to labor-savings and standardization of the new bite taking works.

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Apply and press a wax occlusion rim on the honey comb unit to soften. Standard temperature of 160°C is recommendable to set.

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It is recommendable to move the wax rim in different directions on the honey comb 2~3 times to press.

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When the mandibular base plate touches the honey comb surface, the use of honey comb edge is recommendable.

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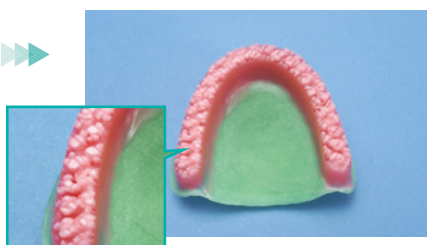
In case of partial rim, we recommend you to remove it from the plaster model.

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【Complete】Softening to an optional depth in uniformity

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Other ways by using occlusal Rim Leveler plate DF-100

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Place a flat rim plate, DF-100, on honey comb unit. Move a wax rim in circles on the plate to make the rim soft and to adjust the height of the rim. Lay a sheet of tissue paper on DF-100 and put down the paraffin wax on there for softening.

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Specification

Power Requirement	AC100V±10% 50/60Hz
Power Consumption	300VA
Temperature Display	Digital display in 3 digits
Temperature Set Range	120~180℃ up/down by 1°C unit measure
Warm-up Time	Approx. 2 min. 30 sec. from room temp. up to 180°C
Measurement・Weight	W160 x D220 x H135mm ・ 1.6kg

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denken CO.,LTD. 130 ISHIGOHRI-CHO OHYAKE YAMASHINA-KU KYOTO 607-8187 JAPAN